

User Guide



Therasage™ Heating Belt

Combining Deep Penetrating Heat Therapy with
“Enriched Air” Negative Ion Technology

ALERT: READ ALL INSTRUCTIONS CAREFULLY BEFORE USING THIS HEALING BELT AND SAVE THEM FOR FUTURE REFERENCE. THIS PRODUCT IS CONSIDERED A MEDICAL DEVICE AND MUST ALWAYS BE USED WITH CARE.

Congratulations on your purchase of our Therasage™ Healing Belt, another of our personal care products designed with your optimal health and wellness in mind.

EASY START UP

Your unique Healing Belt comes to you with easy to wear and care for material, an extension belt to insure a comfortable fit, a small pocket for personal items, a rechargeable lithium battery, a battery recharger, and a carrying bag. To get started, simply remove these items from the box and packing bag, and your Belt is ready for immediate use.

INSTRUCTIONS FOR USE

The key instructions for using your Healing Belt are with the power source, a 7.4 volt Lithium Ion “SMART” battery.

Use only the battery and the battery recharger that is provided with the Healing

Belt. Fully charged, the battery will last approximately four hours, depending on the level of heat used.


Initially recharge the battery ONLY when it has been completely discharged.

To recharge the Lithium Ion SMART battery, you can remove it from its pouch located inside the Healing Belt (or recharge in the pouch) and plug it into the charger. Plug the charger into a standard 110 volt wall socket. Allow up to three hours for the battery to charge, or until the red (“still charging”) light changes to a green (“charged”) light. Unplug the battery from the charger, put the battery into its pouch (located inside the Healing Belt) and plug the power connector from the belt into the battery.

DO NOT OVERCHARGE THE BATTERY.

This will shorten the life of the battery. Always recharge the battery as soon as possible after it becomes fully discharged. A battery will be permanently damaged or impaired if it is left for an extended length of time in a fully discharged state.

Press the green ON/SETUP/ OFF button. Each time you press the button, a corresponding white, green, or orange light will appear indicating if your system is Off, or on Hi or Low.

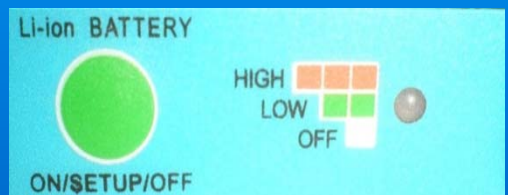

Operating on Hi, the charge will last approximately three hours.
Operating on Low, the charge will last approximately four hours.



WHAT MAKES THIS BELT SO SPECIAL?

EVEN HEAT DISTRIBUTION

Your Healing Belt is uniquely powered by a lithium ion battery to offer therapeutic deep, penetrating Infrared heat. The heat is distributed evenly throughout the back of the Belt by the Advanced Carbon Fiber matrix as well as by the integrated new nano-composite FIR material. This combination is so advanced that heat therapy is available even when the battery is turned off, so you can always *Feel the Heat™*.

Infrared heat is natural energy!

Infrared heat is a safe, natural thermal wave emitted directly by the Sun as part of its invisible energy. The deep, penetrating heat we feel when standing in the sunshine is actually the infrared frequency of the Sun's rays.

Infrared heat promotes health and wellness!

Infrared heat has been clinically proven to reduce minor muscle and joint pain, relax muscles, increase flexibility, improve local blood circulation, decrease harmful toxins, enhance the immune system, reduce stress and fatigue, improve sleep quality, and increase daytime energy and mental alertness.

"ENRICHED AIR" THERAPY THROUGH NEGATIVE ION TECHNOLOGY

Nature's positive mood enhancers, Negative Ions are natural energy that promote health and wellness so you feel great! These invisible charged molecules are found in abundance in nature as moving water interacts with land. Negative Ions are believed to balance our body charge, help calm and relieve stress, produce an overall feeling of wellbeing, and generally effect us in a positive way.

CARE AND SAFETY INSTRUCTIONS

TO CLEAN AND STORE

After each use, the Belt may be wiped with a lightly damp cloth to remove any perspiration or dirt. No solvent or cleaning liquid should be used on the Belt. When cleaning, the temperature of the water used should not exceed 100° F/ 37° C, and the Belt should not be soaked in water.

Place the Heating Belt flat in its box, in its special carrying bag, or in a safe, dry, cool place. Do not tightly fold or crush and avoid contact with sharp or pointed objects that might cut or puncture the Belt. Do not hang or handle the Belt by the battery or battery power cord inside the battery pouch.

Carefully examine the Belt before each use. Do not use if any part of the Belt shows any signs of deterioration such as blistering or cracking; if punctured; if the power cord is pulled away from the Belt, or if the battery is showing any evidence of age.

PROPERLY STORE WHEN NOT IN USE.

SERVICING

ALL SERVICING OF THIS HEALING BELT MUST BE PERFORMED AT AN AUTHORIZED THERASAGE SERVICE CENTER.

WARNING: READ ALL INSTRUCTIONS CAREFULLY BEFORE USING THIS BELT. THIS PRODUCT IS A CLASS II MEDICAL DEVICE AND MUST ALWAYS BE USED WITH CARE. TO REDUCE THE RISKS OF INJURIES OR ACCIDENT, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH ALL INSTRUCTIONS AND PRECAUTIONS.

The U.S. FDA requires us to also state:

"WARNING: USE CAREFULLY. May cause serious burns. Do not use over sensitive skin areas or in the presence of poor circulation. The unattended use of this device by children or incapacitated persons may be dangerous."

PRECAUTIONS

We offer the following specific safety instructions and precautions to help keep our customers accident-free while fully enjoying the benefits of our product.



- This Belt should be used wherever an increase in therapeutic heat application is desired or recommended. To avoid the possibility of skin reaction, rash or burn, we recommend using clothing or a towel barrier between your skin and the Belt.
- Do not use the Belt on skin that is swollen or inflamed, on skin eruptions, or where liniment, essential oils, topical patches, salve or ointment preparations containing heat-producing ingredients have been applied. Skin burns could result.
- If you use any type of medicated patch, consult your personal physician prior to use.
- Supervision is necessary for unconscious individuals, paralyzed persons, or those who have a diminished sensation of cold or heat, and those with limited recognition of their surroundings or any medical condition that would limit the user's capacity to safely operate the Belt.
- This product should not be used in a crib, or by someone under the age of ten without adult supervision. Misuse by a child could cause injury.
- This device should not be used while sleeping.
- **If you are elderly, infirm, pregnant, or if you have a pacemaker or defibrillator implant, diabetes, a blood-based disorder, other medical conditions, or have any concern regarding the use of this product, consult your personal physician prior to use.**

PRECAUTIONS - continued



- Products involving power sources are manufactured with inherent risks. Like all battery-powered devices, the battery should never come in contact with any water, condensation, water source, fire, or high temperatures.
- Healing belts are NOT microwave safe.
- This product is not meant to be taken apart nor fastened with metal pins to hold in place. This product contains a battery-powered component, which, if punctured or tampered with, could cause injury.
- If smoke is ever detected from this device, immediately disconnect the battery.
- Do not position this device near sources of heat or electromagnetic radiation such as radiators, heat registers, stoves, television sets, or magnetized objects.





FREQUENTLY ASKED QUESTIONS

Why is this Belt different than all other belts?

Uniquely powered by both our Active Carbon Fiber matrix, which produces direct deep, penetrating heat, and the self-heating, nano-composite material, which indirectly reacts to the Infrared heat emitted from your body, this integrated Healing Belt maximizes your potential heat therapy. In addition to this heat therapy combination, the Healing Belt is enhanced with Therasage's advanced "enriched air" technology which creates an environment of relaxing and rejuvenating therapeutic Negative Ions.

Is the Healing Belt just for the back?

No. With a flexible format and a unique combination of deep, penetrating heat coming from both the Activated Carbon Fiber as well as the self-heating, nano-composite material, the versatile Belt is suitable for all areas of the body. Just remember to use a towel or clothing barrier between the Belt and your bare skin.

How long should I use the Belt at one time?

As with any new therapeutic device, start off by using it at a moderate heat level for a brief period of time (30-45 minutes). As your body gets used to the therapeutic effects you can gradually increase time and temperature.

Always assert common sense when using the Healing Belt.

You can use the Belt as often and for as long as you like. It is purely personal preference. The more you use it, the more relaxed and revitalized you will feel. However, do not use the Healing Belt while you are sleeping.

FREQUENTLY ASKED QUESTIONS - CONTINUED

Are there any accessories that I should order along with this Healing Belt?

The Healing Belt is a wonderful product. It does not need any other accessories, although many people like to purchase a back-up battery, or a power converter, especially if you travel outside of the country. For additional products, please look at www.therasage.com for the newest items on the market.

Is there anyone who should not use this Healing Belt?

Children ten years old or younger, the elderly, and the infirm. Persons with diseases or disorders associated with reduced blood flow, individuals prone to bleeding, prone to fevers, insensitive to heat, who are pregnant, cardiac patients, and with any medical condition should consult with their health care provider before using this product.

This Belt has been great for long stretches in the car and while sitting on different seats. What other products should I consider?

We invite you to bookmark our website, www.therasage.com, join our Facebook page, and stay tuned for electronic mail notices as we are always bringing new technologies and products to our customers.

On our website, you will find healing pads, saunas, massage beds, and other types of furniture for your home and office, healing wraps and clothing as well as programs and services for Health and Wellness that you might be interested in trying.



Therasage Healing Belt™

Limited Warranty

1. WHAT THIS LIMITED WARRANTY COVERS AND FOR HOW LONG

This Product includes the product itself, and, unless specifically excluded below, all material, parts, and components related to functionality. This Limited Warranty applies only when all of these parts are genuine Therasage components cosmetic components are not covered by this Limited Warranty.

a) COVERAGE

Subject to the limitations and exclusions below, and for the duration of the applicable Warranty Period, Therasage, and any of its affiliates, ("Seller") warrants that the Product's components will be free from material and functional non-conformities in material and workmanship under normal and reasonable consumer conditions.

b) WARRANTY PERIOD

For the first year after purchase, Therasage, at its discretion, will repair or replace any defective part, component, mechanism, or material. This warranty coverage during the first year shall include all defective parts, labor included. The Warranty Period shall commence upon purchase of the product by the end-user consumer from an authorized dealer. Each consumer is urged to register the Product through the Warranty Registration Form attached hereto. This Limited Warranty is not transferable. Finally, this Warranty is effective from the date of purchase as shown on the purchaser's receipt (as registered with the Seller). This Warranty can only commence once proof of purchase (or gift) is provided.

2. WHAT THIS LIMITED WARRANTY DOES NOT COVER

a) GENERALLY

This Limited Warranty does not include non-conformities or damages attributable to or arising from:

- Minor scratches or minor visual imperfections
- Negligence; improper use, installation, finishing, maintenance (including failure to maintain finish) nor operation inconsistent with Seller's recommendations and written instructions that are generally available in Seller's Product Manual as updated by bulletins or other written communications or on Seller's Worldwide website www.therasage.com.
- Improper storage, including inadequate shelter or inadequate venting of shipping wrap in humid locations
- Misapplication of use, commercial use, or use other than generally intended
- Batteries (which are covered under their own warranty)
- Normal wear and tear.

Therasage Healing Belt™

Limited Warranty

b) CERTAIN CONDITIONS

This Limited Warranty does not guarantee health claims. Further, this Limited Warranty does not cover damage attributable to or caused by acts of God that include, but are not limited to, power surges, stresses, fire and other conditions that exceed Product designs. Seller is not responsible for claims or damages caused by water or condensation infiltration.

3. THIS LIMITED WARRANTY'S EXCLUSIVE REMEDY

If the Product or any components fail to meet this Limited Warranty, Seller's sole obligation is to either (as Seller elects): a) repair the component (color matching not guaranteed); or b) provide replacement component(s) to the Warranty Holder or the Seller's dealer designated (color matching not guaranteed); or c) refund the Warranty Holder's purchase price (the lesser of the original Product/component purchase price or the original catalog list price). Repaired or replaced components are warranted only on the same terms and for the remainder of the Warranty Period. Seller reserves the right to discontinue or change any Product. If the Product or component is not available, Seller may select and provide a replacement Product or component of equal quality and price. This is the Warranty Holder's sole and exclusive remedy for the Product under this Limited Warranty. By example but not limitation, this Limited Warranty does not cover the following costs and expenses: (i) labor (after the first year); (ii) shipping/ freight expenses to return the Product to Seller, or Seller's designated service center; (iii) normal maintenance; or (iv) consequential, special, or indirect losses or damages of any kind.

4. DISCLAIMER OF WARRANTIES

THIS LIMITED WARRANTY IS IN LIEU OF AND EXCLUDES ALL OTHER WARRANTIES NOT EXPRESSLY SET FORTH HEREIN, EXPRESSED OR IMPLIED BY OPERATION OF LAW OR OTHERWISE, INCLUDING BUT NOT LIMITED TO ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. TO THE EXTENT THAT ANY IMPLIED WARRANTIES MAY NONETHELESS EXIST BY OPERATION OF LAW, SUCH WARRANTIES ARE LIMITED TO THE DURATION PROVIDED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY. SELLER DOES NOT AUTHORIZE ANYONE TO CREATE FOR IT ANY OBLIGATION OR LIABILITY IN CONNECTION WITH PRODUCTS.

- **Therasage Healing Belt™**
- **Limited Warranty**

5. LIMITATION OF LIABILITY

SELLER'S SOLE LIABILITY UNDER THIS LIMITED WARRANTY IS REPLACEMENT, REPAIR, OR REFUND OF THE PURCHASE PRICE AS SET FORTH ABOVE. IN NO EVENT, WILL SELLER BE LIABLE FOR INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL, OR PUNITIVE DAMAGES INCLUDING, BUT NOT LIMITED TO, DAMAGE OF ANY KIND TO A PREMISES, LOSS OF PRODUCT USE, LABOR, LOST PROFITS/REVENUE, INTEREST, LOST GOODWILL, WORK STOPPAGE, IMPAIRMENT OF OTHER GOODS OR WORK, INCREASED OPERATING EXPENSES, EMOTIONAL DISTRESS CLAIMS, LOSS OF CONSORT CLAIMS, OR CLAIMS OF THIRD PARTIES FOR SUCH DAMAGES, WHETHER BASED ON CONTRACT, WARRANTY, TORT (INCLUDING, BUT NOT LIMITED TO, STRICT LIABILITY OR NEGLIGENCE) OR OTHERWISE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES.) SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY. THIS LIMITED WARRANTY PROVIDES SPECIFIC LEGAL RIGHTS BUT WARRANTY HOLDER MAY HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE. IF THIS LIMITED WARRANTY IS DEEMED TO HAVE FAILED OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL SELLER'S ENTIRE LIABILITY EXCEED THE LESSER OF THE PRODUCT'S OR THE NON-CONFORMING COMPONENT'S PURCHASE PRICE.

6. CLAIMS

Claims must be initiated during the Warranty Period. To initiate a claim, please contact Therasage, Attn: Customer Service, 21000 Boca Rio Road, Suite A-21C, Boca Raton, Florida, 33433 or by writing Info@therasage.com. Claimant will be required to provide proof of purchase and the date of Product purchase (unless the Product was registered at the time of purchase) and may be required to return the Product or component to Seller (at Claimant's expense).

Healing Belt Size*: 45" x 6.75" (114cm x 17cm)
with belt extender 18 " for a total length of 63"

*Actual specifications are subject to change without notice.

THERASAGE SERVICE CENTER

21000 Boca Rio Road
Suite A-21C

Boca Raton, FL 33433-1551

(888) 416-4441 telephone

(888) 416-9991 fax

info@therasage.com

Therasage is dedicated to providing natural, safe, affordable, therapeutic home and office healthcare products. We are guided by our commitment to health and wellness, our pledge to provide customer satisfaction, and our desire to impact positively the environment and society.

While our attorneys remind us that we must state the following, we voluntarily comply with the regulations set forth by federal agencies, most notably by not making claims beyond which we are permitted by law:

The statements contained herein have not been evaluated by the FDA. No medical claims of any kind, directly related to any of our products, are made or implied. Our Therasage products are not intended to diagnose, treat, cure or prevent any disease. The information contained herein is meant to inform and educate. For all medical concerns, please consult your physician or other health care professional.

WE INVITE YOU TO VISIT OUR WEBSITE
www.therasage.com
TO VIEW ALL OF OUR HEALTHY PRODUCTS